PATIENT SLING OWNER'S MANUAL

STANDARD ADULT MODEL NUMBER	DESCRIPTION	EXTRA WIDE MODEL NUMBER
9040	Wrap Around	
9042	Polyester	
9043	Polyester w/Commode Opening	9043W
9044	Polyester w/Headrest	
9045	Polyester w/Commode and Headrest	
9046	Polyester Mesh	
9047	Polyester Mesh w/Commode Opening	9047XW
9047P	Polyester Mesh w/Commode Opening	
9048	Polyester Mesh w/Headrest	
9049	Polyester Mesh w/Commode and Headrest	

NOTE: Slings listed above require a strap or chain kit for hook-up.

9070	Strap Kit	9070
9071	Chain Kit	9071

CHAIN-FREE ALL FABRIC SLINGS

*R110 - R117	Full Body Slings with or without Commode Openings
*R101 - R102	Divided Leg Slings
*R120 - 122	Toileting Slings
*9742	Universal Deluxe Sling
*9746	Hammock Slings with or without Commode Openings
**R140 and R141	Heavy Duty with or without Commode Openings
***R130 and R134	Standing Sling and Transport Sling

* Can ONLY be used with Invacare Models 9700, 9701, 9805, 9806, 9807 and Reliant Series Lifts on the LARGE HOOKS of the swivel bar or on CONVERTED Model Nos. 9801, 9802 and 9803 (9800A Boom Retrofit Kit). The SMALL HOOKS are to be used for CHAIN / STRAP KITS ONLY.

**Can ONLY be used with Invacare Model RPA600 (Adjustable Base) 600 lbs. Lift.

***Can ONLY be used with Invacare Model RPS350-1E Stand Up Lift.

SPECIAL NOTES

WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS; CONTACT INVACARE TECHNICAL SUPPORT OR A QUALIFIED INVACARE DEALER BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

MAINTENANCE

Maintenance MUST be performed ONLY by qualified personnel.

WARNING notices as used in this manual apply to hazards or unsafe practices which could result in serious bodily harm.

CAUTION notices as used in this manual apply to hazards or unsafe practices which could result in minor personal injury or property damage.

NOTES highlight procedures and contain information which assist the operator in understanding the information contained in this manual.

NOTICE

The information contained in this document is subject to change without notice.

**SAVE THESE INSTRUCTIONS
AND
KEEP WITH SLINGS/PATIENT LIFT AT ALL TIMES.**

SAFETY SUMMARY**WARNING**

Check all parts for shipping damages before using. In case of damage, **DO NOT** use the equipment. Contact the Dealer for further instructions.

The Invacare patient lift is **NOT** a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair). Moving a person suspended in a sling over **ANY** distance is **NOT** recommended.

DO NOT attempt any transfer without approval of the patient's physician, nurse or medical attendant. Thoroughly read the instructions in this Owner's Manual, observe a trained team of experts perform the lifting procedures, and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient.

Use common sense in all lifts. Special care **MUST BE** taken with people with disabilities who cannot cooperate while being lifted. Use restraint straps if necessary.

Invacare slings and patient lift accessories are specifically designed to be used in conjunction with Invacare patient lifts. Slings and accessories designed by other manufacturers are not to be utilized as a component of Invacare's patient lift system. Use of these products is prohibited and will void Invacare's patient sling warranty. Use only genuine Invacare slings and lift accessories to maintain patient safety and product utility.

Use a sling that is recommended by the individual's doctor, nurse or medical attendant for the comfort and safety of the individual being lifted.

Before transferring a patient from a stationary object (wheelchair, commode or bed), slightly raise the patient off the stationary object and check all sling hardware for secure attachment to include swivel bar, straps, S-hooks and D or O-rings. If any attachment is not correct, lower the patient and correct the problem, then raise the patient and check again.

During transfer, with patient suspended in a sling attached to the lift, **DO NOT** roll caster base over objects such as raised carpet bindings, door frames, or any uneven surfaces or obstacles that would create an imbalance of the patient lift and could cause the patient lift to tip over. Use steering handles on the mast at **ALL** times to push or pull the patient lift.

Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard Immediately.

DO NOT alter slings.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and sling material that may cause the patient to slide out of the sling during transferring.

ALWAYS use the color coded strap on the standing sling closest to the patient while still maintaining patient stability and comfort.

STANDING SLINGS - DO NOT use the standing sling in combination with the stand up lift as a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair). Moving a person using the standing sling in combination with the stand up lift over **ANY** distance is **NOT** recommended.

STANDING SLINGS - Before lifting the patient, make sure the bottom edge of the standing sling is positioned on the lower back of the patient and the patient's arms are outside the standing sling.

STANDING SLINGS - The belt **MUST** be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer, possibly causing injury.

TRANSPORT SLINGS - Before lifting the patient, make sure the bottom edge of the transport sling is at the base of the spine and the patient's arms are outside the transport sling.

ALWAYS use the color coded strap on the standing sling closest to the patient while still maintaining patient stability and comfort.

DO NOT exceed maximum weight limitation of the patient lift. These suggested weights will vary from 300 to 450 to 600 lbs. depending on the type of patient lift purchased. For maximum weight limitation, refer to the **SPECIFICATIONS** section in the Portable Patient Lift and Sling Owner's Manual or contact Customer Service, 1-(800)-333-6900.

DO NOT exceed maximum weight limitation of the Stand Up Lift. The maximum weight limitation is 350 lbs. regardless of any additional weight limitations on accessories.

STANDARD SLINGS

MODEL NUMBERS		DESCRIPTIONS
STD.	EXTRA WIDE	
9042		Polyester
9043	XW	Polyester w/Commode Opening
9044		Polyester w/Headrest
9045		Polyester w/Commode and Headrest
9046		Polyester Mesh
9047	XW	Polyester Mesh w/Commode Opening
9047P		Polyester Mesh w/Commode Opening
9048		Polyester Mesh w/Headrest
9049		Polyester Mesh w/Commode and Headrest

Invacare slings are designed specifically for use with Invacare Patient Lifts and made to support the patient during lift and transfer procedures. Slings attach to the lift with either adjustable straps or chains (interchangeable). The slings are reinforced at all points of attachment for safety. The fabrics used in these slings are of very durable materials that resist deterioration from exposure to moisture and laundering. This design will accommodate patients of heavy or light weight with support from the shoulders to the knees. The contour shape provides comfort during the lift by avoiding the bulk of the excess material between the back and the seat sections.

Fabric

The fabric has a smooth inner side. The outer side has binding tape around the pocket.

NOTE: Always place the sling under the patient with the smooth side next to the body.

Sling fabrics are offered in a choice of two (2) materials. Polyester fabric is very durable for use and laundering and is preshrunk and dyed a blue color.

Open polyester mesh fabric is an exceptional material that provides safety and comfort for the patient. The open mesh weave of the fabric is dyed, preshrunk and durable for use and laundering. It provides considerable traction to resist slipping and sliding during use.

NOTE: Laundering should always be done with dark colors.

WARNING
After each laundering, inspect sling(s) for wear, tears, and loose stitching.

Body

Slings may be ordered at the time of purchase with a commode opening in the seat section for use over a toilet or a commode chair. Slings may also be ordered with a headrest for additional upper body support.

Each sling includes four (4) metal support bars that have a hole at the center to attach the “S” hook.

When shipped from the factory, these reinforcing metal support bars are enclosed in the sling pocket with the hole exposed.

Positioning the Sling (WITHOUT Headrest) (FIGURE 1)

1. Position the patient in the center of the bed and laying flat on his/her back.
2. Fold the sling in half (length-wise) and place the fabric sections of the sling beside the patient.

FIGURE 1 - POSITIONING OF THE STANDARD SLING

NOTE: The sling's french seams (uneven surface) will be against the bed surface and facing the attendant when folded. The opposite side of the fabric is smooth. After the sling is positioned for lifting purposes, the smooth side will be next to the patient.

3. The top edge of the sling fabric (nearest to the patient's head) should be slightly above the armpit level of the patient.
4. The bottom edge of the sling fabric should then be a few inches above the back of the patient's knee.
5. With both attendants holding the patient, push the folded sling under the patient without rolling him/her over.

Positioning the Patient on the Sling (WITHOUT Headrest)

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

1. If the patient is to roll to their LEFT-SIDE, then elevate the patient's RIGHT KNEE until the right foot is flat on the bed (**FIGURE 2**).

NOTE: Invacare recommends that two (2) attendants (One [1] on each side of the bed) be used when positioning the patient in a sling.

2. With an attendant on each side of the bed and up against the mattress, the attendant on the left-hand side of the bed will position his/her RIGHT-HAND on the elevated KNEE and his/her LEFT-HAND under the patient's RIGHT SHOULDER, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side (**FIGURE 2**).
3. With the patient on their side, push the fabrics of the seat and back gently under them (**FIGURE 2**).

NOTE: The upper edge of the back section should be positioned slightly above the armpit level and the lower edge of the seat section positioned a few inches above the back of the patient's knees (**FIGURE 1**).

4. Roll the patient on to his/her back (**FIGURE 3**).

NOTE: Attendants will reverse roles.

5. After the patient has been positioned once again on his/her back, you now need to roll the patient to their RIGHT-SIDE (facing the attendant on the LEFT).
6. With an attendant on each side of the bed and up against the mattress, the attendant on the right-hand side of the bed will elevate the LEFT KNEE and position his/her LEFT-HAND on the elevated KNEE and his/her RIGHT-HAND under the patient's LEFT SHOULDER, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side (**FIGURE 3**).
7. Pull the fabrics of the seat and back across the mattress until they are smooth (**FIGURE 3**).

FIGURE 2 - ROLLING PATIENT ONTO HIS/HER SIDE

STANDARD SLINGS

8. Roll the patient onto their back and they should be approximately centered on the sling section (**FIGURE 3**).
9. You are now ready to attach the sling to the chain or strap.

Positioning the Standard Sling (WITH Headrest) (FIGURES 4 and 5)

1. Open the snap buttons on the back of the sling and insert the metal support into the opening on each side and secure under the fabric with the snap buttons (**FIGURE 4**).

FIGURE 3 - ROLLING PATIENT ONTO HIS/HER OPPOSITE SIDE AND BACK

FIGURE 4 - HEADREST SLING AND PATIENT POSITIONING

NOTE: The metal support **MUST** be positioned in the sling angled towards the patient's head for proper support. The metal support can **NOT** be positioned in the sling angled away from the patient's head. This will permit his/her head to fall backwards.

Positioning the Standard Sling (WITH Headrest)

NOTE: Invacare recommends that two (2) attendants (One [1] on each side of the bed) be used when positioning the patient in a sling.

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

1. Position the patient in the center of the bed and laying flat on his/her back (**FIGURE 4**).
2. If the patient is to roll to their LEFT-SIDE, then elevate the patient's RIGHT KNEE until the right foot is flat on the bed (**FIGURE 4**).
3. With an attendant on each side of the bed and up against the mattress, the attendant on the left-hand side of the bed will position his/her RIGHT-HAND on the elevated KNEE and his/her LEFT-HAND under the patient's RIGHT SHOULDER, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side.
4. Fold the top and bottom french seams of the sling fabric (**FIGURE 5**).

NOTE: There will be a limit to folding because of the stiffness of the headrest.

5. With the patient on their side, push the fabrics of the seat and back gently under them (**FIGURE 5**).

NOTE: The sling's french seams (uneven surface) will be against the bed surface and facing the attendant when folded. The opposite side of the fabric is smooth. After the sling is positioned for lifting purposes, the smooth side will be next to the patient.

NOTE: The patient's head should be positioned in the headrest just below the metal support for maximum comfort.

NOTE: The upper edge of the back section should be positioned at the armpit level and the lower edge of the seat section positioned a few inches above the back of the patient's knees.

6. Roll the patient on to his/her back.

NOTE: Attendants will reverse roles.

7. After the patient has been positioned once again on his/her back, you now need to roll the patient to their RIGHT-SIDE (facing the attendant on the LEFT).

FIGURE 5 - ROLLING PATIENT ONTO HIS/HER OPPOSITE SIDE AND BACK

8. With an attendant on each side of the bed and up against the mattress, the attendant on the right-hand side of the bed will elevate the LEFT KNEE and position his/her LEFT-HAND on the elevated KNEE and his/her RIGHT-HAND under the patient's LEFT SHOULDER, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side.
9. Pull the fabrics of the seat and back across the mattress until they are smooth.
10. Roll the patient onto their back and they should be approximately centered on the sling section.
11. You are now ready to attach the sling to the chain or strap.

MODEL NO. 9040 WRAP-AROUND SLING

MODEL NO. 9040 WRAP-AROUND SLING (FIGURES 6A and 6B)

NOTE: Headrest is not available on the 9040 Wrap-Around Sling.

NOTE: Invacare recommends that two (2) attendants (One [1] in front and one [1] in back of the wheelchair) be used when positioning the patient in the wrap-around sling.

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

WARNING

If the patient is in a wheelchair, secure the wheel locks in place to prevent the chair from moving forward or backwards.

1. With the patient sitting in a chair, one (1) attendant in the front and the other attendant in the back, lean the patient forward.


NOTE: The front attendant will be supporting the weight of the patient.

2. Place the wrap-around sling behind the patient and bring the flaps out alongside the patient's legs.

NOTE: The back of the wrap-around sling (mounting area) should be parallel to the patient's upper arms and be positioned between the top of the patient's back and the top of the chair back.

3. With the back of the sling positioned properly, push the edges of the commode opening under the patient's buttocks.
4. Lean the patient back into the chair with the attendant in the rear supporting his/her weight.
5. Lift the patient's legs (one at a time) and holding the snap hook of the sling, reach under the patient's leg and pull until the front of the sling is behind the patient's knees (about three [3] inches) and the back of the sling remains in position.

6. Wrap the leg extension and snap hook around the leg and snap onto the "D" hook. Repeat this step for the other leg.
7. You are now ready to attach the sling to the chains or straps.

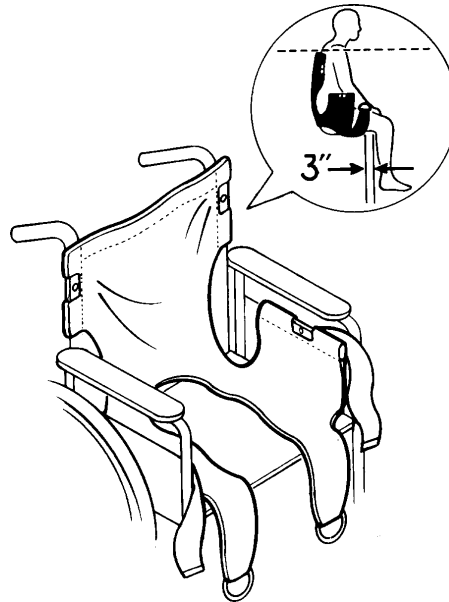**FIGURE 6A - POSITIONING THE PATIENT IN THE
MODEL 9040 - WRAP-AROUND SLING**

FIGURE 6B - POSITIONING THE PATIENT IN THE MODEL 9040 - WRAP-AROUND SLING

ATTACHING THE SLING HARDWARE

MODEL NO.:

9070 - Strap Kit (Can be used with Models 9040 - 9049 ONLY)

9071 - Chain Kit (Can be used with Models 9040 - 9049 ONLY)


The hardware used with the standard sling includes four (4) metal support bars. The support bars insert into the pockets at the edge of the fabric. The pocket that will hold the metal bar has an opening in the center. The pocket is sewn closed on one end and has an open-end with a strap on the other.

NOTE: When the slings are laundered, remove the four (4) metal support bars. They MUST be replaced or the sling can not be used for lifting purposes.

Replacing the Four (4) Metal Support Bars in the Sling (FIGURES 1A AND 1B)

NOTE: When inserting the support bar into the sling pocket, align the metal support bar in the center of the strap (after it has been pulled down) and then push the metal support bar into the sling pocket. The sling is sewn on both sides so the center location is critical.


1. To insert the metal support bar, pull the strap down allowing the bar to pass through the opening in the sling pocket.
2. Push the metal support bar through until the mounting hole clears the center opening of the sling pocket.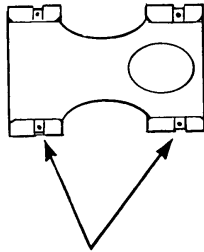

Metal Support Bars with Openings for Chains or Straps

3. Once the opposite end of the bar is inside the sewn pocket, push the strap over the metal support bar to prevent it from coming out.

NOTE: The hole in the metal bar should be in the center of the pocket opening. This is where the "S" hook of the chain or strap will be attached.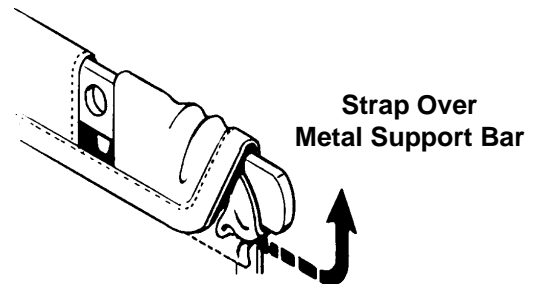

Strap Over Metal Support Bar

FIGURE 1A - REPLACING THE FOUR (4) METAL SUPPORT BARS IN THE SLING

FIGURE 1B - REPLACING THE FOUR (4) METAL SUPPORT BARS IN THE SLING

Attaching the Chains or Straps to the Slings (FIGURE 2)

NOTE: Each sling will work with either a pair of chain assemblies or a pair of strap assemblies.

CAUTION

When using either the chains or straps to connect the sling to the patient lift, the shortest of the two (2) sections of the chains or straps **MUST** be attached to the back section of the sling. Using the long section will leave little or no support for the patient's back.

1. With the patient laying on his/her back and positioned properly in the sling, lay the chains or straps on the bed next to the sling (FIGURE 2).

2. The short section of chain or strap will be placed parallel to the patient's back. This will position the longer section of chain or strap from the waist to the mid section of the thigh.
3. The "S" hooks on the ends of the chains or straps are inserted into the metal support bar of the sling.
4. The "S" hook **MUST** be inserted through the hole in the support bar with the open end facing away from the patient (FIGURE 2).
5. Position the lift for use. Refer to **POSITIONING LIFT FOR USE** in the **LIFTING PREPARATION** section of this manual.

NOTE: The lift must be positioned for use before the slings can be attached.

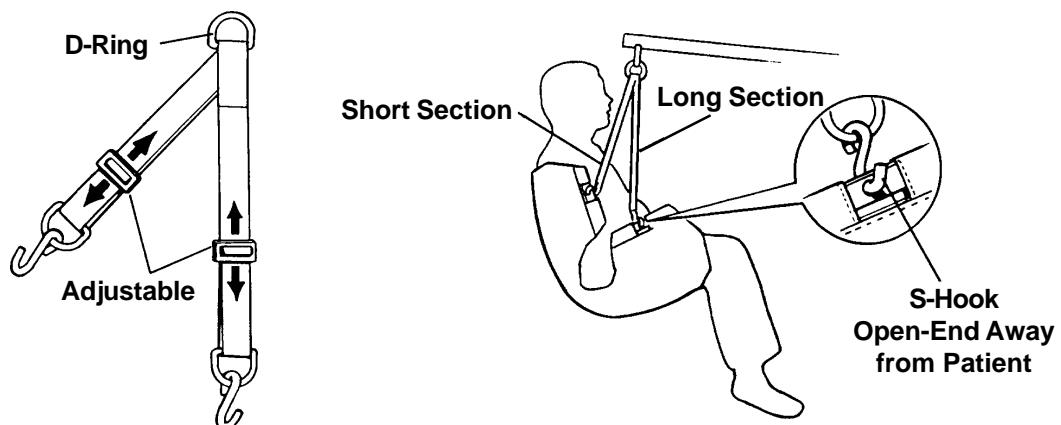
FIGURE 2 - ATTACHING THE HARDWARE TO THE SLING

FULL BODY, DIVIDED LEG AND TOILETING SLINGS

MODEL NOS. R110 - R117 FULL BODY WITH OR WITHOUT COMMODE OPENING, MODEL NOS. R100 - R102 DIVIDED LEG AND MODEL NOS. R120 - 122 TOILETING SLINGS (FIGURE 1)

Invacare slings are designed specifically for use with Invacare Patient Lifts and made to support the patient during lift and transfer procedures. Slings attach to the lift with color coded straps for easy positioning. The slings are reinforced at all points of attachment for safety. The fabrics used in these slings are of very durable materials that resist deterioration from exposure to moisture and laundering. Designs will accommodate patients of heavy or light weight with full support from the head to the knees.

Fabric

The outer side has a reinforced "grab handle" for assistance in positioning the patient in a wheelchair, commode, etc. The edging is soft yet durable with extra reinforced stitching for safety.

NOTE: Always place the sling under the patient with the handle away from the patient.

NOTE: Laundering should always be done with dark colors. Refer to tagged washing instructions on the sling.

Open mesh or solid knit polyester fabric is an exceptional material that provides safety and comfort for the patient. The open mesh weave of the fabric is dyed, preshrunk, and durable for use and laundering. It provides considerable traction to resist slipping and sliding during use.

Body

Slings may be ordered at the time of purchase with a commode opening in the seat section for use over a toilet or a commode chair. Slings are constructed with a high back section that provides additional support for the head and neck and a padded under thigh section for comfort.

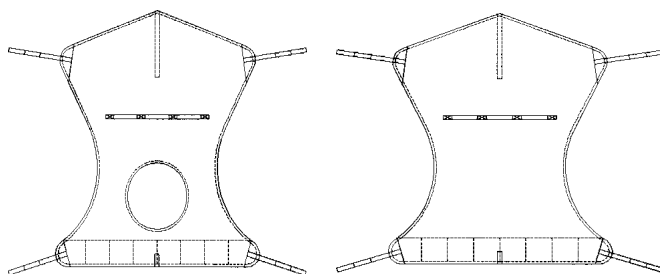
FIGURE 1 - MODEL NOS. R110 - R117 FULL BODY SLINGS WITH OR WITHOUT COMMODE OPENING

MODEL NOS. R100 -102 DIVIDED LEG AND R120 - 122 TOILETING SLINGS (FIGURE 2)

Divided leg design is best suited for high level spinal cord injuries or patients which sling removal is more difficult. Support ranges from the upper back to beneath the thighs. Can be positioned with patient seated. Greatly simplifies the use of restroom facilities.

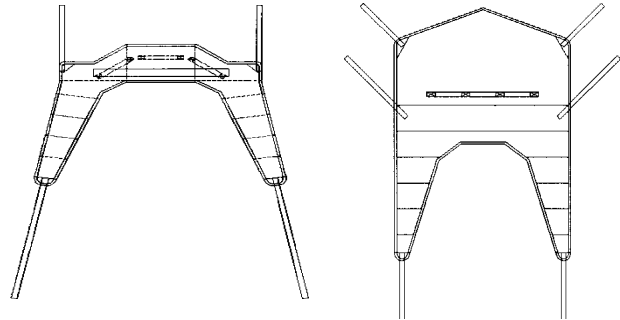
FIGURE 2- MODEL NOS. R100-102 DIVIDED LEG AND R120 - 122 TOILETING SLINGS

MODEL NO. R140 - R141 HEAVY DUTY SLINGS (FIGURE 3)

Full body mesh sling with optional commode opening. Two (2) straps per side. Comfortably supports up to 600 lbs.

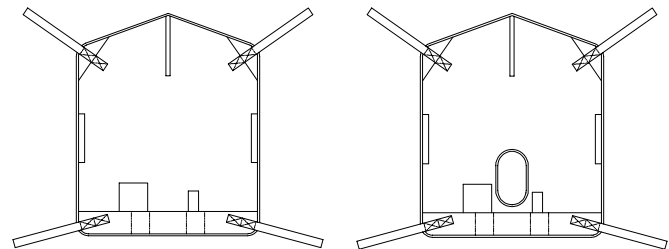
FIGURE 3 - MODEL NO. R140 - R141 HEAVY DUTY SLINGS

FULL BODY, DIVIDED LEG AND TOILETING SLINGS

POSITIONING THE MODEL NOS. R110 - R117 FULL BODY SLINGS WITH OR WITHOUT COMMODE OPENINGS (FIGURE 4)

1. Position the patient in the center of the bed and laying flat on his/her back.
2. Fold the sling in half (length-wise) and place the sling beside the patient.

NOTE: The closed-end or commode opening of the sling with positioning handle should be facing the patient when folded.

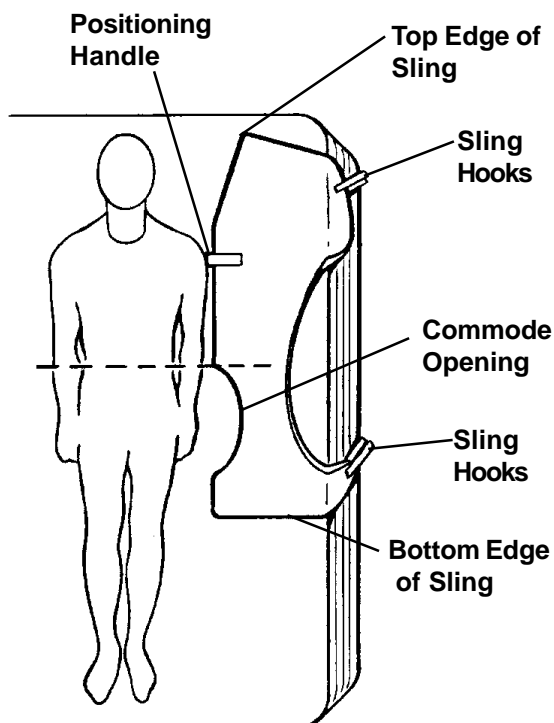
FIGURE 4 - POSITIONING THE SLING

3. The top edge of the sling fabric should be slightly above the patient's head.
4. The bottom edge of the sling fabric should then be a few inches above the back of the patient's knees.

NOTE: Invacare recommends that two (2) assistants (One [1] on each side of the bed) be used when positioning the patient on a sling. The bed rails may also be raised to minimize patient movement.

5. With one (1) assistant holding the patient, the second assistant pushes the folded sling under the patient without rolling him/her over.

Positioning the Patient on the Sling with or without Commode Opening

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

1. If the patient is to roll to their LEFT-SIDE, then elevate the patient's RIGHT KNEE until the right foot is flat on the bed (FIGURE 5).

FIGURE 5 - ROLLING PATIENT ONTO HIS/HER SIDE

FULL BODY, DIVIDED LEG AND TOILETING SLINGS

NOTE: Invacare recommends that two (2) assistants (One [1] on each side of the bed) be used when positioning the patient onto a sling.

- 2. With an assistant on each side of the bed and up against the mattress, the assistant on the left-hand side of the bed will position his/her RIGHT-HAND on the elevated KNEE and his/her LEFT-HAND under the patient's RIGHT SHOULDER, slowly push on the knee and assist with a slight lift of the shoulder. The patient will easily roll onto their side (FIGURE 5).
- 3. With the patient on their side, push the fabrics of the seat and back gently under them (FIGURE 5).

NOTE: The patient's head should be positioned in the headrest just below the top edge for maximum comfort and the lower edge of the seat section positioned a few inches above the back of the patient's knees (FIGURE 4).

- 4. Roll the patient on to his/her back (FIGURE 6).

NOTE: Assistants will reverse roles.

- 5. After the patient has been positioned once again on his/her back, you now need to roll the patient to their RIGHT-SIDE (facing the assistant on the LEFT).
- 6. With an assistant on each side of the bed and up against the mattress, the assistant on the right-hand side of the bed will elevate the LEFT KNEE and position his/her LEFT-HAND on the elevated KNEE and his/her RIGHT-HAND under the patient's LEFT SHOULDER, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side (FIGURE 6).
- 7. Pull the fabrics of the seat and back across the mattress until they are smooth (FIGURE 6).
- 8. Roll the patient onto their back and they should be approximately centered on the sling section (FIGURE 6).
- 9. Position the lift for use. Refer to **POSITIONING LIFT FOR USE** in the **LIFTING PREPARATION** section of this manual.

NOTE: The lift must be positioned for use before the slings can be attached.

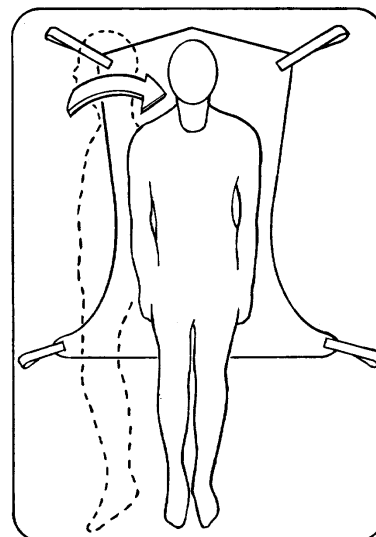
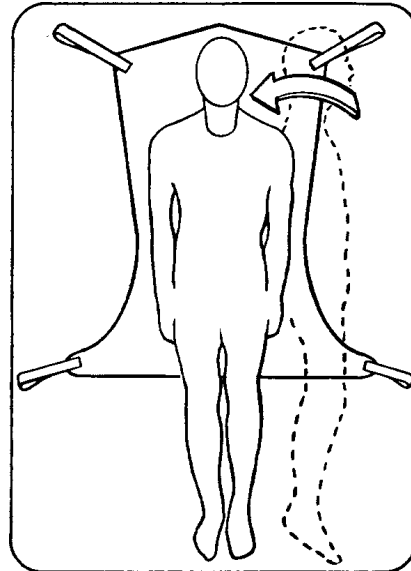
FIGURE 6 - ROLLING PATIENT ONTO HIS/HER OPPOSITE SIDE AND BACK

DIVIDED LEG / TOILETING SLING

**MODEL NOS. R100 - R102 DIVIDED LEG AND
MODEL NOS. R120 - 122 TOILETING SLINGS
(FIGURES 7A and 7B)**

Positioning the Patient in the Divided Leg Sling

NOTE: Invacare recommends that two (2) assistants (one [1] in front and one [1] in back of the wheelchair) be used when positioning the patient in the divided leg sling.

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

WARNING

If the patient is in a wheelchair, secure the wheel locks in place to prevent the chair from moving forward or backwards.

1. The rear wheel locks of the wheelchair are locked to prevent movement of the chair.
2. With the patient sitting in a chair one (1) assistant in the front and the other assistant in the back, lean the patient forward.

NOTE: The front assistant will be supporting the weight of the patient.

- 3a. Place the divided leg sling behind the patient (with "grab handle" on the outside) and bring the flaps out alongside the patient's legs.
- 3b. Toileting slings have a waist buckle which helps hold the sling in place during positioning.

NOTE: For the **DIVIDED LEG SLING:** The back of the sling should be parallel to the patient's upper arms and be positioned between the top of the patient's head and the base of the spine.

For the TOILETING SLING: The back of the sling should be parallel to the patient's upper arms and be positioned between the top of the patient's mid-back and the base of the spine.

4. With the back of the sling positioned properly, push the bottom of the back section to the seat.
5. Lean the patient back into the chair with the assistant in the rear supporting his/her weight.
6. Lift the patient's legs (one at a time) and reach under the patient's leg and pull until the front of the sling is behind the patient's knees (about 3-inches) and the back of the sling remains in position.

NOTE: The straps may be **AROUND**, **crossed BETWEEN** or **UNDERNEATH** the patient's legs.

7. Position the lift for use. Refer to **POSITIONING LIFT FOR USE** in the **LIFTING PREPARATION** section of this manual.

NOTE: The lift must be positioned for use before the slings can be attached.

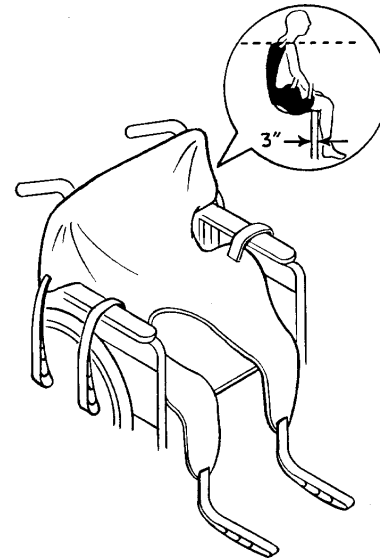
FIGURE 7A - POSITIONING THE PATIENT IN THE DIVIDED LEG / TOILETING SLINGS

DIVIDED LEG / TOILETING SLING

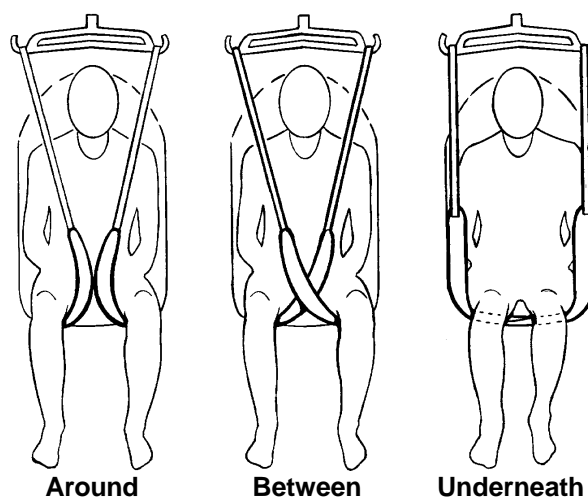
FIGURE 7B - POSITIONING THE PATIENT IN THE DIVIDED LEG / TOILETING SLINGS

LIFTING PREPARATION

POSITIONING LIFT FOR USE (FIGURE 1)

NOTE: Refer to **PATIENT LIFT SAFETY SUMMARY** in the front of this manual before proceeding further and observe all **WARNINGS** indicated.

WARNING

Before lifting or transferring the patient, the base legs **MUST** be **LOCKED** in the **OPEN** position for optimum stability and safety.

NOTE: Before positioning the legs of the patient lift under a bed, make sure that the area is clear of any obstructions.

1. With the legs of the base **OPEN** and **LOCKED**, use the steering handle to push the patient lift into position.
2. Lower the patient lift for easy attachment of the sling.
3. Refer to **ATTACHING SLINGS TO LIFT** in this section of the manual.

ATTACHING SLINGS TO LIFT

Slings Using Chains or Straps (FIGURE 2)

1. The D-ring (strap) or O-ring (chain) is the suggested location to attach the swivel bar, however, by changing to different links or adjusting the strap, you can reposition the patient into a reclining or more upright position.

WARNING

When changing to different lengths, **DO NOT** disconnect the **O-rings** of the chains from the swivel bar. Position **BOTH** the **O-rings** and additional chain link onto the hooks of the swivel bar.

2. The straps can be simply adjusted to achieve the same effect.
3. With the sling positioned under the patient, and the support bars in place, you are now ready to move the lift into position to attach the sling hardware to the swivel bar.
4. Refer to the **LIFTING THE PATIENT** Section in the Portable Patient Lift and Sling Owner's Manual.

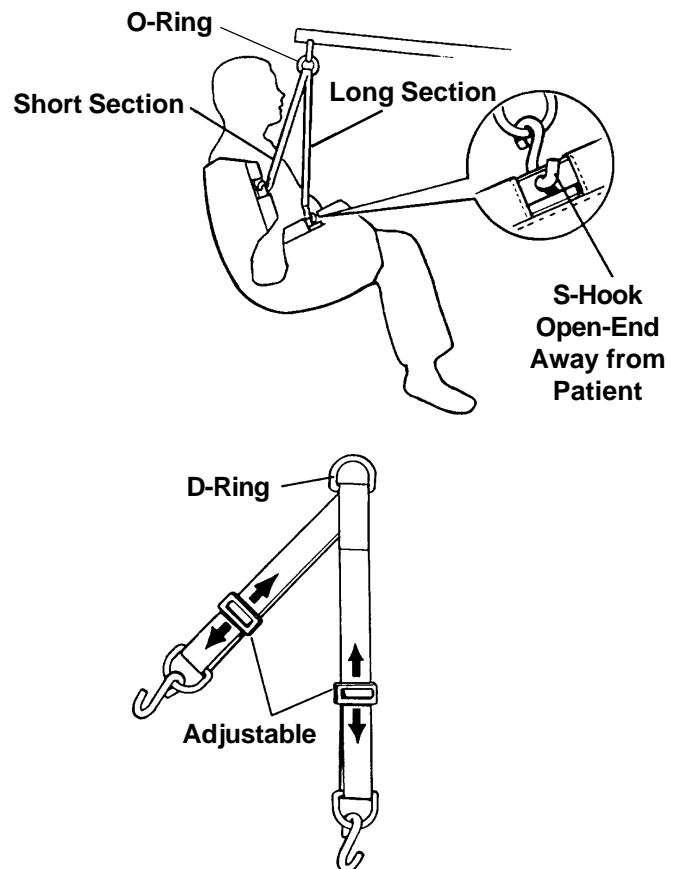
FIGURE 2 - ATTACHING SLINGS TO LIFT - SLINGS USING CHAINS OR STRAPS

LIFTING PREPARATION

Slings Using Color Coded Straps (FIGURE 3)

1. Place the straps of the sling over hooks of the swivel bar. Match the corresponding colors on each side of the sling for an even lift of the patient (FIGURE 3).

CAUTION

When connecting sling to the patient lift, the shortest of the straps **MUST** be at the back of patient for support. Using long section will leave little or no support for patient's back. The loops of the sling are color coded and can be used to place patient in various positions. The colors make it easy to connect both sides of the sling equally. Make sure that there is sufficient head support when lifting a patient.

NOTE: Model Nos. R110 - R117 Full Body Slings and Model Nos. 120 - 122 Toileting Slings have four (4) sling straps. Model Nos. 100 - 102 Divided Leg Slings have six (6) sling straps.

2. Refer to the **LIFTING THE PATIENT** Section in the Portable Patient Lift and Sling Owner's Manual.

NOTE: Sling Model Nos. 9746, 9742 and Reliant Slings can **ONLY** be used with PATIENT LIFT Model Nos. 9700, 9701 and Reliant Series Lifts, or with the **LARGE HOOKS** of the swivel bar on PATIENT LIFT Model Nos. 9805, 9806, 9807, **CONVERTED** Model Nos. 9801, 9802 and 9803 (9800A Boom Retrofit Kit). The **SMALL HOOKS** are to be used for **CHAIN/STRAP KITS ONLY**.

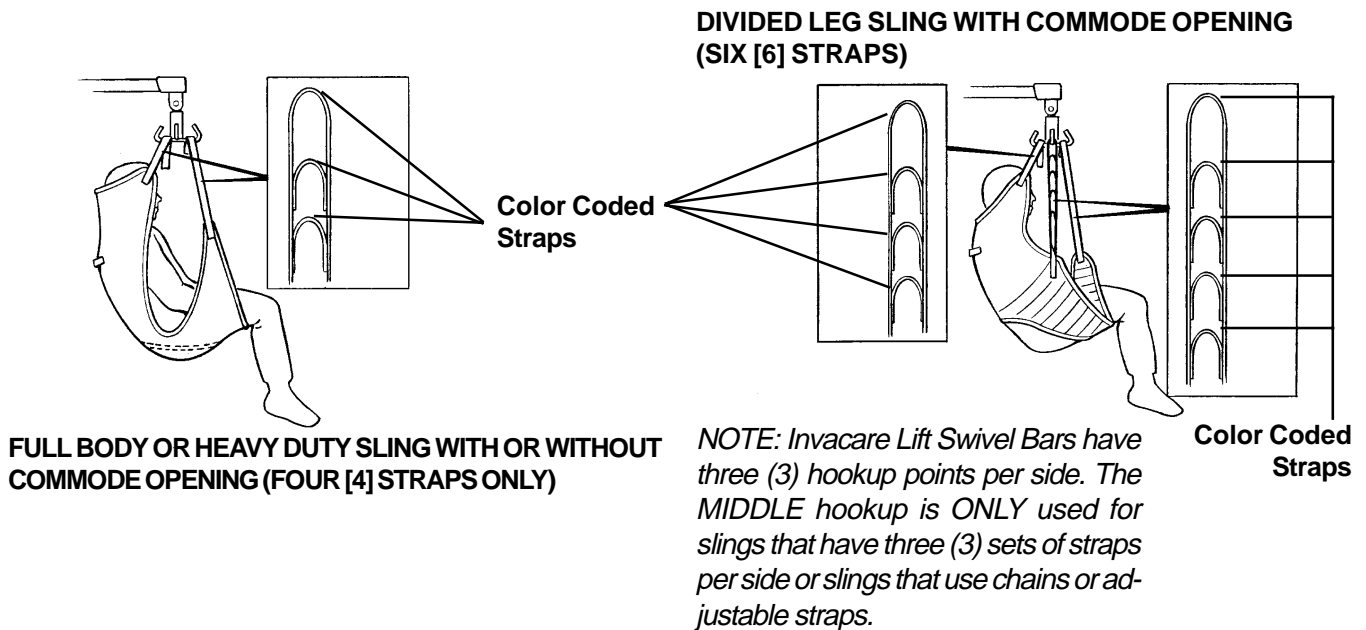
FIGURE 3 - ATTACHING SLINGS TO LIFT - SLINGS USING COLOR CODED STRAPS

STANDUP AND TRANSPORT SLINGS

PATIENT SLINGS (FIGURE 1)

WARNING

Invacare standing and transport slings are specifically designed to be used in conjunction with the Invacare Stand Up Lift. Slings and accessories designed by other manufacturers or other Invacare slings are not to be utilized as a component of Invacare's stand up lift system. Use of these products is prohibited and will void Invacare's stand up lift warranty. Use only genuine Invacare standing and transport slings and lift accessories to maintain patient safety and product utility. Use the sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual being lifted.

Read sling label for compatible lift usage BEFORE using ANY sling/Stand Up Lift combination.

Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard Immediately.

DO NOT alter slings.

Standing (Model R130) and Transport (Model R134) attach to the lift with color coded straps. The color coded straps allow for adjustment.

Standing (Model R130) and Transport (Model R134) slings are constructed of durable materials which resist deterioration from exposure to moisture or laundering.

NOTE: Laundering should always be done with dark colors. Refer to tagged washing instructions on the sling.

Standing Sling - Model R130

WARNING

Individuals that use the Standing Sling (Model R130) **MUST** be able to support the majority of their own weight, otherwise injury can occur.

Standing slings assist individuals that can support the majority of their own weight in a standing position. Multiple hook up points adjust to body sizes.


Transport Sling - Model R134

WARNING

DO NOT raise the patient to a full standing position while using the transport sling (model R134), otherwise injury can occur.

Transport slings transfer fully dependant individuals in a seated position. Transport slings hook under the patient's legs and support their torso. Multiple hook up points adjust to body sizes.

STANDING SLING - MODEL R130 (BACK VIEW OF SLING)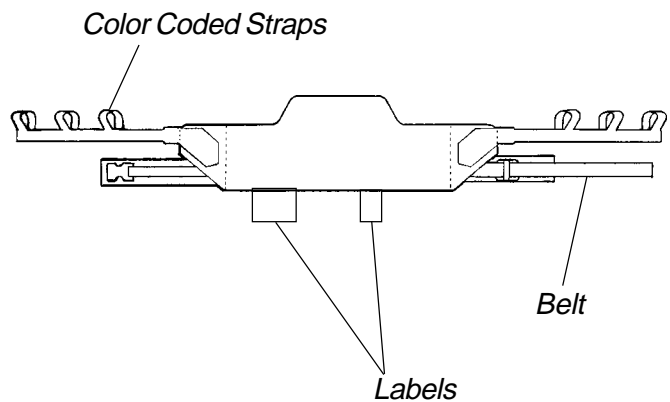

TRANSPORT SLING - MODEL R134 (BACK VIEW OF SLING)

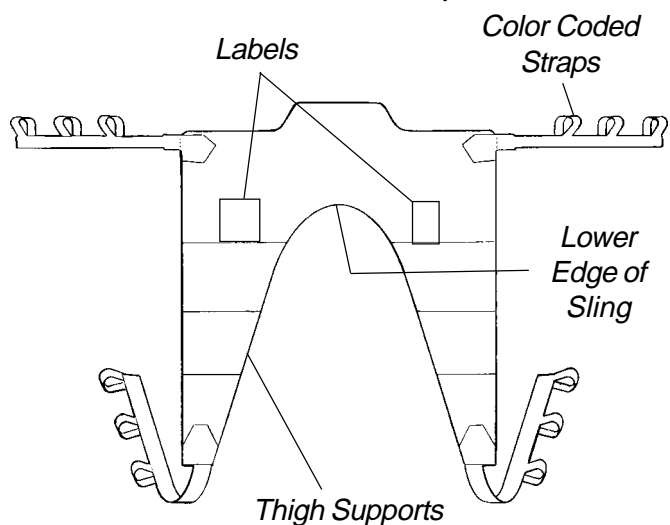
FIGURE 1 - PATIENT SLINGS

LIFTING PREPARATION

WARNING

Invacare recommends that two (2) assistants be used for ALL lifting preparation, transferring from and transferring to procedures.

NOTE: Assist the patient if necessary.

4. Continue to push the stand up lift around the patient until the patient's knee are secure against the knee pad.

POSITIONING THE STAND UP LIFT AROUND PATIENT (FIGURE 1)

STEPS 2 AND 3

NOTE: Refer to the SAFETY SUMMARY in the front of this manual before proceeding further and observe all WARNINGS indicated.

WARNING

DO NOT attempt any transfer without approval of the patient's physician, nurse or medical assistant. Thoroughly read the instructions in the Owner's Manual P/N 1078796, observe a trained team of experts performing the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient.

STEP 4

The legs of the stand up lift **MUST** be in the **MAXIMUM OPEN** position and the shifter handle **LOCKED** in place for optimum stability and safety. If the patient is in a transport sling and it becomes necessary to move through a narrow passage, close the legs of the stand up lift **ONLY AS LONG AS IT TAKES TO MOVE THROUGH THE PASSAGE**. Once the stand up lift is through the passage, return the legs of the stand up lift to the **MAXIMUM OPEN** position and **LOCK** shifter handle **IMMEDIATELY**.

Before positioning the legs of the stand up lift around the patient, make sure the patient's feet are out of the way of the footplate, otherwise injury can occur.

STEP 5

NOTE: If preparing to lift the patient from a bed, make sure there is nothing under the bed before pushing the stand up lift under the bed and around the patient.

1. **OPEN** and **LOCK** the legs of the base. Refer to **CLOSING/OPENING THE LEGS OF THE BASE** in the **OPERATION** Section of Owner's Manual P/N 1078796.
2. Using the push handle, begin pushing the stand up lift around the patient. Make sure the patient's feet are out of the way of the footplate.
3. Have the patient position their feet on the footplate as shown in **FIGURE 1**.

FIGURE 1 - POSITIONING THE STAND UP LIFT AROUND PATIENT

WARNING

Invacare recommends locking the rear swivel casters **ONLY** when positioning or removing the sling (standing or transport) from around the patient.

5. Lock the rear swivel casters. Refer to **LOCKING/UNLOCKING THE REAR SWIVEL CASTERS** in the **OPERATION** Section of Owner's Manual P/N 1078796.
6. Lower the stand up lift all the way down for easy attachment of either the standing sling (Model R130) or the transport sling (Model R134). Refer to **RAISING/LOWERING THE STAND UP LIFT** in the **OPERATION** Section of Owner's Manual P/N 1078796.
7. Position either the stand up sling or the transport sling around the patient. Refer to **POSITIONING THE SLINGS AROUND THE PATIENT** in this section of the manual.

POSITIONING THE SLINGS AROUND THE PATIENT

Standing Sling (Model R130) (FIGURE 2)

WARNING

Individuals that use the Standing Sling (Model R130) **MUST** be able to support the majority of their own weight, otherwise injury can occur.

DO NOT use the standing sling in combination with the stand up lift as a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair). Moving a person using the standing sling in combination with the stand up lift over **ANY** distance is **NOT** recommended.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and sling material that may cause the patient to slide out of the sling during transferring.

Before lifting the patient, make sure the bottom edge of the standing sling is positioned on the lower back of the patient and the patient's arms are outside the standing sling.

1. Position the standing sling around the patient's torso just below the shoulder blades and under the arms.

NOTE: Make sure the bottom edge of the sling is on the lower back and the patient's arms are outside the standing sling.

NOTE: The belt should be towards the patient with the labels at the bottom of the standing sling.

2. Secure the standing sling to the patient by buckling the belt.

WARNING

The belt MUST be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer, possibly causing injury.

3. Adjust the belt for a snug, but comfortable fit.
4. Attach the standing sling to the stand up lift. Refer to **ATTACHING THE SLINGS TO THE STAND UP LIFT** in this section of the manual.

STEP 1

STEP 2

STEP 3

FIGURE 2 - POSITIONING THE SLINGS AROUND THE PATIENT - STANDING SLINGS

Transport Sling (Model R134) (FIGURE 3)

STEP 1

WARNING

DO NOT raise the patient to a full standing position while using the transport sling (model R134), otherwise injury can occur.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and sling material that may cause the patient to slide out of the sling during transferring.

Before lifting the patient, make sure the bottom edge of the transport sling is at the base of the spine and the patient's arms are outside the transport sling.

1. Position the transport sling behind the patient as shown in **STEP 1** of **FIGURE 3**. Make sure the bottom edge of the sling is at the base of the spine.

NOTE: The labels on the transport sling should be up and facing away from the patient.

2. Lift up one (1) of the patient's legs and position the thigh support under the leg.
3. Repeat **STEP 2** for the opposite thigh support.
4. Pull the thigh support of the transport sling under each thigh of the patient until the thigh supports reach completely around the thigh's of the patient.

NOTE: Make sure the transport sling fabric is smooth underneath the thigh's of the patient.

5. Attach the transport sling to the stand up lift. Refer to **ATTACHING THE SLINGS TO THE STAND UP LIFT** in this section of the manual.

STEPS 2, 3 AND 4

**FIGURE 3 - POSITIONING THE SLINGS
AROUND THE PATIENT - TRANSPORT SLINGS**

ATTACHING THE SLINGS TO THE STAND UP LIFT

Standing Sling (Model R130) (FIGURE 4)

1. Determine the best attachment point for the standing sling on each lift arm. Use the following chart as a guideline to determine attachment point.

NOTE: The following chart is for reference ONLY.

ATTACHMENT POINT	PATIENT HEIGHT RANGE
Short	under 5' 6"
Medium	5' 6" to 5' 10"
Tall	over 5' 10"

WARNING

ALWAYS use the color coded strap on the standing sling closest to the patient while still maintaining patient stability and comfort.


Make sure the color coded strap sits flush against the attachment knob in STEP 3C of FIGURE 4, otherwise injury or damage can result.

NOTE: The color coded straps can be used to place the patient in various positions. The colors make it easy to connect both sides of the sling equally.

2. From the OUTSIDE of the stand up lift, attach the standing sling to the stand up lift at the attachment point determined in **STEP 1**.
3. Make sure of the following:
 - A. The standing sling is wrapped around the OUTSIDE of each lift arm.
 - B. Use the closest color coded strap to the patient that maintains stability and comfort.
 - C. The color coded strap sits flush against the attachment knob.
 - D. The same color coded strap is used on each side of the standing sling.
 - E. The same attachment point is used on each lift arm.


NOTE: If necessary, adjust the attachment point of the transport sling to each lift arm.

4. Once a stable and comfortable position is found for the patient, mark the attachment points and color coded strap used in the patient file for future use.
5. Transfer the patient. Refer to one (1) of the following procedures in the Standup Lift Owner's Manual P/N 1078796:
 - A. TRANSFERRING FROM
 - B. TRANSFERRING TO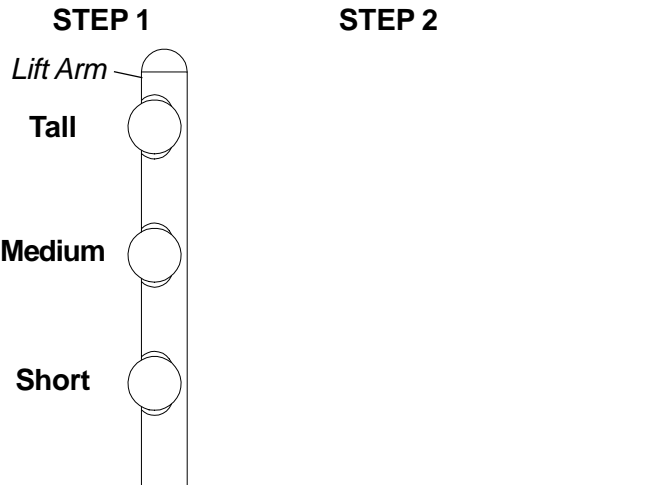

STEP 3E

STEPS 3A AND 3B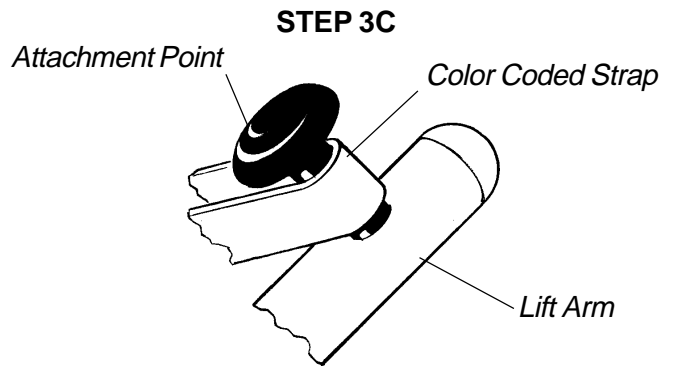

STEP 3D

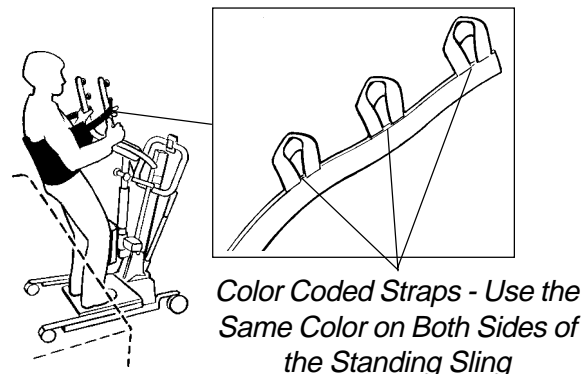
FIGURE 4 - ATTACHING THE SLINGS TO THE STAND UP LIFT - STANDING SLINGS

STANDUP AND TRANSPORT SLINGS

Transport Sling (Model R134) (FIGURE 5)

1. Determine the best attachment point for the transport sling on each lift arm. Use the following chart as a guideline to determine attachment point.

NOTE: The following chart is for reference ONLY.

ATTACHMENT POINT	PATIENT HEIGHT RANGE
Short	under 5' 6"
Medium	5' 6" to 5' 10"
Tall	over 5' 10"

WARNING

ALWAYS use the color coded strap on the transport sling closest to the patient while still maintaining patient stability and comfort.

Make sure the color coded strap sits flush against the attachment knob in STEP 4C of FIGURE 5, otherwise injury or damage can result.

NOTE: The color coded straps can be used to place the patient in various positions. The colors make it easy to connect both sides of the sling equally.

2. Pull the thigh supports forward and attach to the lower attachment points just above the pivoting knee pad.
3. Attach the top portion of the transport sling to each lift arm at the attachment point determined in **STEP 1**.
4. Make sure of the following:
 - A. The transport sling is wrapped around the **OUT-SIDE** of each lift arm.
 - B. Use the closest color coded strap to the patient that maintains stability and comfort.
 - C. The colored coded strap sits flush against the attachment knob.
 - D. The same **top** color coded strap is used on each side of the transport sling.
 - E. The same attachment point is used on each lift arm.
 - F. The same **bottom** color coded strap is used on each side of the transport sling.

NOTE: The top and bottom colored straps do not have to be the same.

NOTE: If necessary, adjust the attachment point of the transport sling to each lift arm.

5. Once a stable and comfortable position is found for the patient, mark the attachment points and color coded strap used in the patient file for future use.
6. Transfer patient. Refer to one (1) of the following procedures in the Standup Lift Owner's Manual P/N 1078796:
 - A. **TRANSFERRING FROM**
 - B. **TRANSFERRING TO**

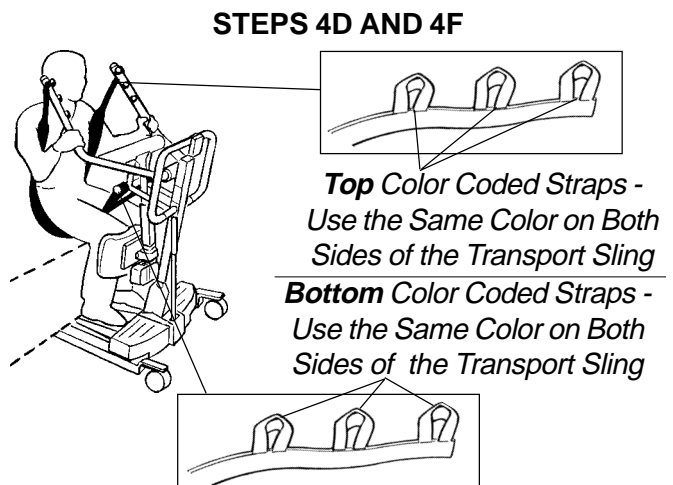
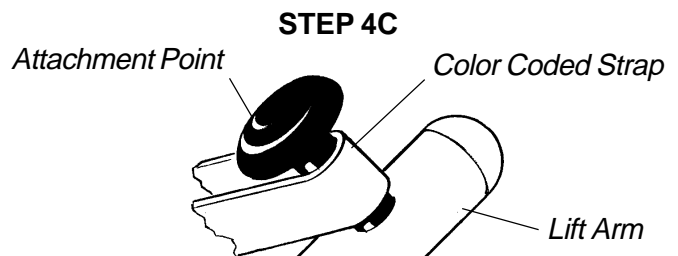
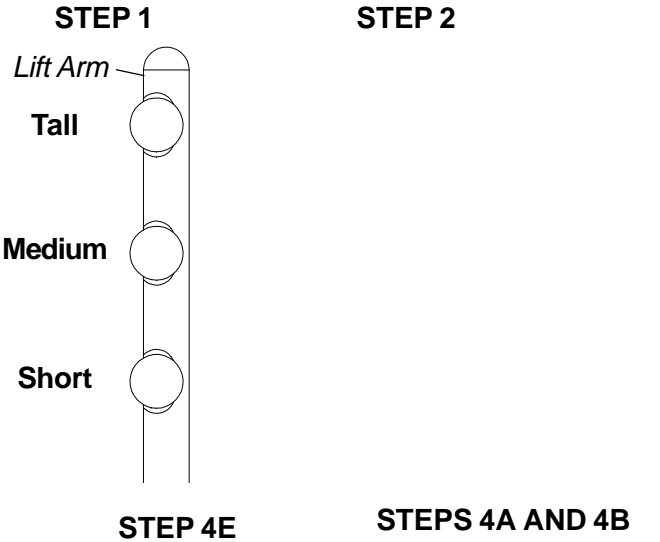
FIGURE 5 - ATTACHING SLINGS TO THE STAND UP LIFT - TRANSPORT SLINGS

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the products manufactured to be free from defects in material and workmanship for a period of one (1) year from the date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, the date of purchase, indicate nature of the defect.

Invacare Corporation will issue a serialized return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable as identification within 30 days of return authorization date. Do not return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTIONS IN THIS MANUAL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE OR PROVINCIAL LAWS AND REQUIREMENTS.

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**Invacare Canada Inc. • 5970 Chedworth Way •