# RELIANT SCALE RLS6

## Assembly, Installation and Operating Instructions

Yes, you can!

### SAVE THESE INSTRUCTIONS

*NOTE: The Reliant Scale CANNOT be used in conjunction with the swivel bar supplied with the Invacare 9805 Lift. The Reliant Swivel Bar (Part No. 1064518) MUST be purchased and installed on the Invacare 9805 lift in order to install the Reliant Scale.*

*NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer/Carrier for further instructions.*

## SAFETY SUMMARY

### GENERAL WARNINGS

**DO NOT** install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

The weight capacity is limited to the lowest rated capacity of any one of the components in use (i.e. Patient Lift, Sling or Scale). The patient's weight **MUST NOT** exceed the lowest rated capacity of any component.

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely - otherwise injury or damage may occur.

Patient and sling **MUST** be removed from the lift during **ALL** installation procedures.

This kit contains the following:

| DESCRIPTION                                   | QTY |
|---|-----|
| Load Cell Assembly                            | 1   |
| Nylon Washers                                 | 2   |
| Swivel Bar Pin                                | 1   |
| Locknut                                       | 1   |
| Mounting Screw                                | 1   |
| Small Steel Washer                            | 1   |
| 1/8-inch thick nylon washer (9805 lift)       | 2   |
| 1/4-inch thick nylon washer (450/600 lb lift) | 2   |

---

## DESCRIPTION

The Reliant Scale is a compact precision scale system designed specifically for the Invacare Patient Lift System.

| SPECIFICATIONS               |   |
|------------------------------|---|
| <b>Weight Range:</b>         | Up to 600 lbs. (272.7 kg.)                |
| <b>Resolution:</b>           | +/- 0.2 lbs (.1 kg)                       |
| <b>Display:</b>              | Liquid Crystal Digital                    |
| <b>Automatic Power Down:</b> | Two (2) minutes                           |
| <b>Size:</b>                 | 5-3/4 L x 3-5/8 W x 4 H                   |
| <b>Weight:</b>               | 1.55 lbs                                  |
| <b>Power:</b>                | Nine (9) volt alkaline battery (Included) |
| <b>Battery Life:</b>         | Approximately 1500 readings               |
| <b>Temperature Range:</b>    | 50° to 104° F (10° to 40° C)              |

## REMOVE THE SWIVEL BAR (FIGURE 1)

- I. Perform one (1) of the following:
  - A. FOR 450 lb and 600 lb RELIANT LIFTS - Remove shoulder bolt, locknut, pinch guard and washer which secure the swivel bar to the boom mounting bracket.

*NOTE: Save the shoulder bolt, small steel washer and locknut to secure the Reliant Scale to the boom. The pinch guard will not be used when the scale is installed. Save pinch guard for future use of swivel bar without scale.*


- B. FOR MODEL NO. 9805 LIFT - Remove shoulder bolt and locknut which secure the swivel bar to the boom mounting bracket.

*NOTE: Save the shoulder bolt and locknut to secure the Reliant Scale to the boom.*


*NOTE: Removing the swivel bar hardware will release the swivel bar, two (2) nylon washers and the swivel bar pin. Save the swivel bar pin and the two (2) nylon washers for future use of the swivel bar without the scale.*

### 450 LB / 600 LB RELIANT LIFTS

*\*NOTE: 600 lift swivel bar not shown.*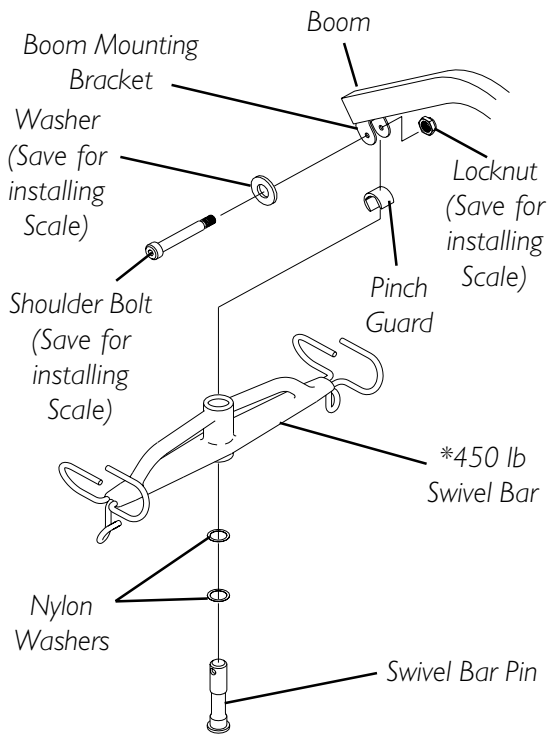

### 9805 LIFT

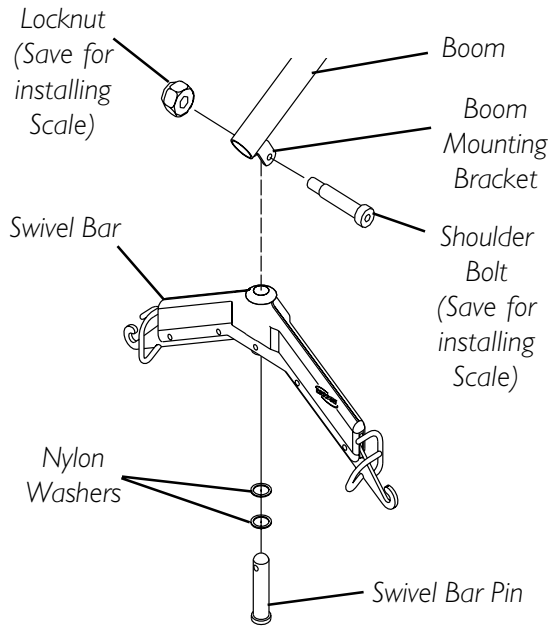*NOTE: Some hardware has been enlarged for clarification purposes.*

**FIGURE I - REMOVE THE SWIVEL BAR**

## INSTALLING THE RELIANT SCALE (FIGURE 2)


1. Position the load cell assembly of the Reliant Scale into the boom mounting bracket. Refer to DETAIL "A" in FIGURE 2.

*NOTE: For 9805 lift use 1/8-inch thick nylon washer. For 450 lb/600 lb lifts use 1/4-inch nylon washer.*

2. Secure the Reliant Scale to the boom mounting bracket with a shoulder bolt, one (1) small steel washer, two (2) nylon washers and a locknut. Securely tighten. Refer to DETAIL "A" in FIGURE 2 for washer orientation.

*NOTE: Ensure the shaft of the shoulder bolt passes through both sides of the boom mounting bracket.*

3. Insert swivel bar pin with two (2) nylon washers through the swivel bar. Refer to DETAIL "B" in FIGURE 2.
4. Align the mounting holes in the swivel bar pin with the mounting holes in the load cell assembly. Refer to DETAIL "B" in FIGURE 2.
5. Secure swivel bar pin to the load cell assembly with the provided mounting screw and locknut. Securely tighten. Refer to DETAIL "B" in FIGURE 2.
6. Weigh the patient. Refer to OPERATING THE SCALE in this instruction sheet.

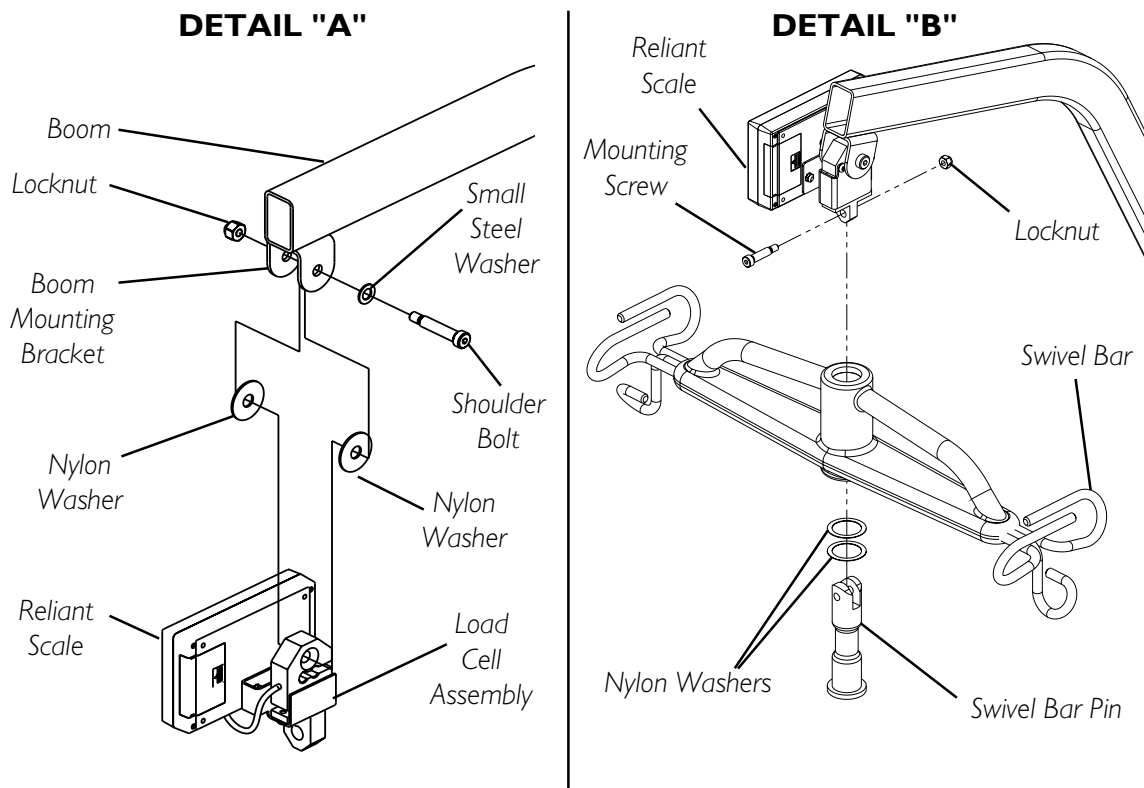**FIGURE 2 - INSTALLING THE RELIANT SCALE**


# OPERATING THE SCALE

## KEYPAD FUNCTIONS (FIGURE 3)

### CAUTION

**DO NOT** operate key strokes with pointed objects (i.e. pencils, pens, fingernails, etc.) - otherwise damage to the key pad will result.

| KEY                   | INDICATOR DISPLAYED | INDICATOR LOCATION                       | DEFINITION   |
|-----------------------|---------------------|--|--|
| <u>ON</u><br>OFF      | OFF                 | Center of Display Window                 | Pressing this key will apply power to the scale and turn the unit on. When the scale is already on, pressing the button will turn the unit off.  |
| ZERO                  | ZERO                | Lower Left Corner of the Display Window  | Pressing this key when the scale is on will reset the weight shown in the display window to zero.  |
| UNITS                 | lb or kg            | Upper Right Corner of the Display Window | The function of this key is to change the unit of measurement from pounds (lb) to kilograms (kg).  |
| <u>LOCK</u><br>UNLOCK | LOCK                | Lower Right Corner of the Display Window | This key is used to LOCK or UNLOCK a weight value in the display window.<br><i>NOTE: Weight displayed is a STORED weight and not the current weight when the key is used in the lock position.</i> |
| N/A                   | LO BAT              | Center of Display Window                 | Indicator is shown in the display window to notify when battery is low.  |

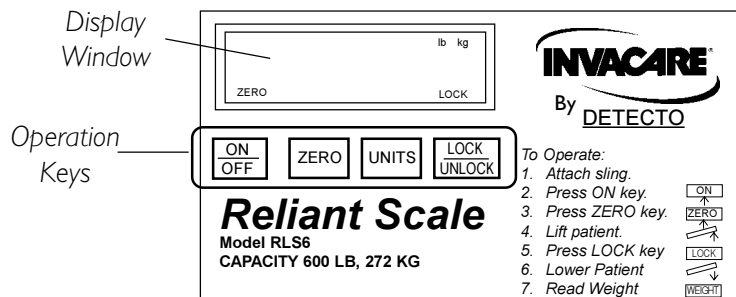**FIGURE 3 - KEYPAD FUNCTIONS/OPERATING THE SCALE**

---

## WARNING

**Invacare slings and patient lift accessories are specifically designed to be used in conjunction with Invacare patient lifts. Slings and accessories designed by other manufacturers are not to be utilized as a component of Invacare's lift system.**

**Use of these products is prohibited and will void Invacare's patient lift warranty.**

**Use only genuine Invacare slings and patient lift accessories to maintain patient safety and product utility.**

**Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).**

1. Attach sling straps to the swivel bar. For proper attachment instructions refer to the Sling Owner's Manual - Part No. 1023891.

*NOTE: For improved accuracy on the 450 and 600 Reliant lifts, attach all sling straps to the center hooks on both sides of the swivel bar. However, patient comfort may require a different configuration of sling straps.*

2. Press the ON/OFF key.

*NOTE: The display will indicate the last weight that was measured. The word "LOCK" will be seen in the display box.*

*NOTE: The ZERO key is pressed in order to avoid capturing the weight of the sling and the hardware. If the ZERO key is not pressed the weight of the sling and the weight of the hardware will be included in the weight displayed. NOT ZERO-ING OUT WILL GIVE A FALSE READING OF THE USER'S TRUE WEIGHT.*

3. Press the ZERO key. When the ZERO key is pushed the following will happen:
  - A. The scale will reset to zero and the word "ZERO" will appear in the display.
  - B. The lock function will then be turned off and the word "LOCK" will disappear from the display.

*NOTE: The scale is now active and continually updating the weight display.*

4. Place the patient in the sling. For patient placement instructions refer to the Sling Owner's Manual - Part No. 1023891.

## WARNING

**When elevated a few inches off the surface of a stationary object (bed, chair or commode) and before moving the patient, check to make sure that the sling is properly connected to the hooks of the swivel bar. If any attachments are NOT properly in place, lower the patient back onto the stationary surface and correct this problem - otherwise, injury or damage may occur.**

5. Activate the lift mechanism to raise the patient until they are completely supported by the lift.

- Note the weight display.
- When the weight display becomes stable press the LOCK button to lock the weight display. This will be indicated by the word "LOCK" appearing in the display window.

*NOTE: Should it be necessary to unlock the weight while the patient is still supported by the lift, the UNLOCK button may be pressed. The weight will unlock and the word "LOCK" will disappear from the display window. The weight value will then be updated. Once the weight becomes stable the weight can be locked again by pressing the LOCK button.*

*NOTE: Stable being defined as the weight fluctuating two tenths (2/10) of a pound. For example, a patient weighing one hundred (100) lbs the scale will fluctuate between 99.8 and 100.2 until the LOCK key is pressed. Fluctuation of the weight displayed is normal as noted above. Press the LOCK button to lock the weight.*

*NOTE: The UNITS button may be pressed to toggle between units of pounds and kilograms. This is indicated by lb or kg appearing in the display window.*

- The lift may now be lowered and the sling removed from the patient.

**NOTES:**

*The patient's weight will continue to be seen in the display window.*

*The display will turn off automatically after a two minute period of non-use [no changes in weight exceeding five (5) pounds (2 kilograms)]. You can NOT adjust the time delay for automatic shut off.*

*After the display has turned off, the weight may be recalled by pressing the ON/OFF button.*


*The unit can be turned off by pressing the ON/OFF button a second time.*

## **BATTERY REPLACEMENT (FIGURE 4)**

*NOTE: The scale is powered by a nine (9) volt alkaline battery that should provide approximately 1500 readings before needing replacement.*

When battery replacement is needed, **LO BAT** will appear on the display. Perform the following:

- Slide the battery door open in the direction of the arrow.
- Remove existing battery.
- Install NEW battery.
- Reinstall the battery door.

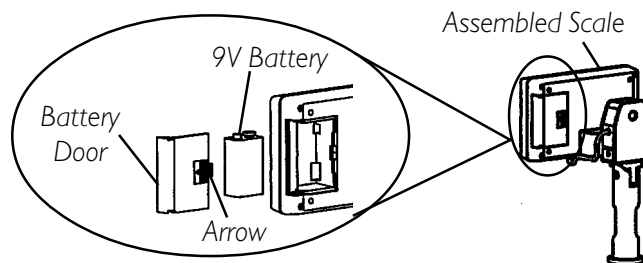**FIGURE 4 - BATTERY REPLACEMENT**

---

## **CALIBRATION OF THE RELIANT SCALE**

*NOTE: The Reliant Scale will be pre-calibrated at the factory with the load cell. Should it be necessary to re-calibrate the scale follow the instructions outlined below.*

1. The patient and the sling must be removed from the scale to properly calibrate the Reliant Scale. For removing the patient instructions refer to the Sling Owner's Manual - Part No. 1023891. For sling detachment instructions refer to the Sling Owner's Manual - Part No. 1023891.
2. With the Reliant Scale on, remove the four screws on the back of the enclosure and remove front cover to expose the PC board.
3. Press the CAL button located on the PC board. The CAL switch is not labeled but is the only button on the PC board located in the lower right corner. Once the CAL button is pushed "CAL1" will be seen in the display window.

*NOTE: CAL1 selects the calibration mode using 50 pounds of calibrated weight. CAL2 selects the calibration mode for use with 200 pounds of calibrated weight. Pressing the UNITS key toggles between CAL1 and CAL2.*

4. When the desired calibration mode is displayed, press the LOCK/UNLOCK key.

*NOTE: The display window will now show UnLd.*

5. Ensure that there is no load on the scale and press the ZERO key.
6. The display window will show a dashed line (-----) scrolling across and then the word LOad will appear in the display window.
7. Perform one (1) of the following:
  - A. FOR CAL1 OPTION - support 50 pounds of calibrated weight from the scale and press the ZERO key.
  - B. FOR CAL2 OPTION - support 200 pounds of calibrated weight from the scale and press the ZERO key.
8. The display window will show a dashed line (-----) scrolling across and then the word dOnE will appear in the display window.
9. Remove the weight from the scale and press the LOCK/UNLOCK key.

*NOTE: The unit will now be in a temporary test mode and will not lock the weight display. This will allow weight to be loaded and unloaded to check the calibration.*

10. Turn the unit off by pressing the ON/OFF key.
11. Turn the unit on by pressing the ON/OFF key.

*NOTE: Scale will now be in normal operation .*

---

## TROUBLESHOOTING

| SYMPTOM  | PROBABLE CAUSE   | SOLUTIONS                                      |
|--|------------------|--|
| Unit does NOT work properly.                                     | Battery failure. | Check battery. Replace if necessary.           |
| Battery has been replaced and unit still does NOT work properly. |                  | Contact Invacare for Service at 1-800-333-6900 |

## DISPLAY CODES

**CAL**

CALIBRATION REQUIRED - Indicates improper stored calibration data, calibration is necessary.

**OCAP**

OVER CAPACITY - Indicates a weight exceeding the capacity has been loaded on the scale.

---

## **LIMITED WARRANTY**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the Reliant Scale to be free from defects in materials and workmanship for a period of two (2) years from the date of purchase. If within such warranty period the Reliant Scale shall be proven to be defective, the Reliant Scale shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the bottom of the page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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© 2003 Invacare Corporation
Invacare is registered trademark of
Yes, you can is a trademark of Invacare